



# IKAZIA 5921



Shown: with wall mount  
(art.no. 7.25.926.00)

## CHARACTERISTICS

- Eye wash bottle including eye bath
- Flushing fluid consists of a sterile saline solution:  
0.9% Sodium Chloride (NaCl)
- Essential first aid item
- To be used if chemicals, dirt or splinters get into the eye
- The eye wash prevents or reduces damage to the eye
- Equipped with an integrated and ergonomic shaped eye bath so that the bottle can be properly aimed at the eye and the eyes remain open during rinsing
- Easy to operate with one hand
- Compact and hygienic
- The vial is for single use and with the contents can be rinsed intensively for about five minutes
- The storage and use temperature of this eyewash is between 5°C and +35°C
- Content: 500 ml

**Article number: 7.25.921.00**

## SUITABLE FOR ACTIVITIES IN E.G.

- Construction
- Petrochemical industry
- Laboratories
- Metal industry
- Transport & logistics
- Automotive

## PACKAGING

- 1 piece
- 10 pieces in outer box

**CE 0123**

### PRODUCT INFORMATION

SIZE	ARTICLE NO.	EAN CODE 1 ITEM	EAN CODE 10 ITEMS (OUTER BOX)
One size	7.25.921.00	8718249079924	8718249079931

### INSTRUCTIONS FOR USE

Use to thoroughly rinse out dirt in the eye or to clean small wounds on the skin.

- Carefully twist the cap in both directions to break the seal and remove the cap.  
Place the bottle above the eye and squeeze gently so that the liquid flows freely over the eyeball to the outer corner of the eye. Clean the eye or eyes with as much solution as necessary.

### INSTRUCTIONS FOR USE WITH EYE BATH

- Open the bottle as described above. Place the eye bath directly on the eye and squeeze gently so that the liquid can flow freely over the eye. Clean the eye with as much solution as necessary.

### PLEASE NOTE!

There are no known side effects associated with the use of sterile sodium chloride solution 0.9% w/v. Any unpleasant effects should be reported to a doctor, pharmacist, or nurse.

**Any incident involving the eyes must be taken seriously. Seek medical attention immediately after use.**



- For external use only
- Not suitable for injection
- Do not use in intraocular surgery
- Do not use if the solution changes color or becomes cloudy



- Do not reuse; discard unused contents after opening
- Avoid contamination
- Do not use as contact lens solution
- Use before the expiration date



### RELATED ARTICLES



#### OXXA LAURENTIUS 5926 EYE WASH BOTTLE WALL MOUNT

Art. no. 7.25.926.00

### YOUR SUPPLIER: